

Directions for use :

Do not expose the patient eye to any topical medication or any manipulation prior to performing the test. Place the patient eye away from direct light, adequately resting the head. Open the strip pouch as indicated in the adjoining figures.



Peel of the pouch using the thumbs and the index fingers of both hands.

Pull open the pouch to about half its length.



Gently remove the strip, holding only the flat end.

Moistened the colored tip of the strip with sterile water or ophthalmic solution. Insert the Moistened end of the strips into the lower conjunctival sac, while the patient looks up. Hold the strip until adequate dye migrates to the patient eye. For optimal result, the patient should blink several times after application.

Fluorescein Sodium Ophthalmic Strips U.S.P.



100 STRIPS
STERILE INDIVIDUALLY PACKED

Steile: Unless enclosed package has been opened or damaged.



Composition:

Each strip contains
1mg Fluorescein Sodium U.S.P.



8 904278 700131

FLUORESCEIN SODIUM OPTHAMIC STRIPS

For visualize defects or abrasions in the corneal epithelium by staining the areas of cellular loss.

For evaluating contact lens fitting & applanation tonometry.

Available in individually wrapped sterile pouches.

Each strip contains approximately 1mg of Fluorescein Sodium IP



aadpri

AADPRI OPTHALMIC COMPANY

G-15, Mahavir krupa complex, Near C.P. Nagar Cross road,
Bhuyangdev char rasta, Ahmedabad- 380061, Gujarat, India.
Mobile no: +91 9374862303,
Email: aadpri@gmail.com Web: www.aadpri.com